Instructions

NL EN FR

Create account & activate kit

Saliva collection

Send back

Open here
to start

Start

Activering van de DNA-speekselkit

Activation of the DNA saliva kit by registering the barcode
Activation du kit ADN salive par enregistrement du code barre

1

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- EN This is what you need: Funnel (A), saliva tube (B), safety bag (C).
Precautions for purchase:

- The sample collection kit is intended for single use.
- Do not use the kit if the packaging is damaged or if the expiration date has been exceeded. The expiration date can be found on the tube.
- Only open the test kit just before use.
- Wait 30 minutes after eating, drinking, smoking or chewing gum before collecting saliva.

Most people will take 2 to 5 minutes to obtain a saliva sample according to steps 5-10.

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- EN Remove the test tube and funnel from the return box.

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
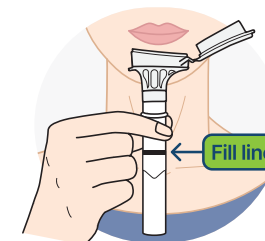
- EN Spit into the funnel until the amount of liquid saliva (without bubbles) reaches the fill line as shown in Figure 1.

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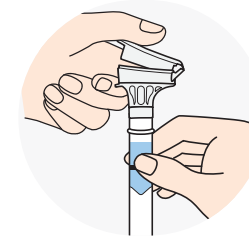- EN Hold the tube upright with one hand. Close the funnel lid with your other hand (as shown) by pressing the lid firmly until you hear a loud click. The liquid in the lid is released into the tube, after which it is mixed with the saliva. Make sure the lid is closed properly.

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- EN Hold the tube upright. Unscrew the funnel from the tube.

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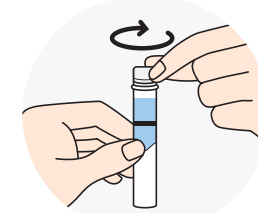
- EN Close the tube tightly with the cap.

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
- EN Shake the tube with the cap on for 5 seconds. You can throw away or recycle the funnel.

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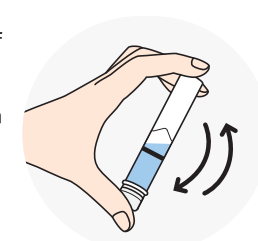
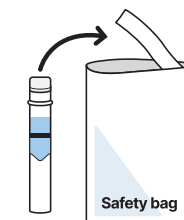
- EN Place the collected samples in the safety bag. After you've made sure everything is in there, pull off the white adhesive edge and close the bag.

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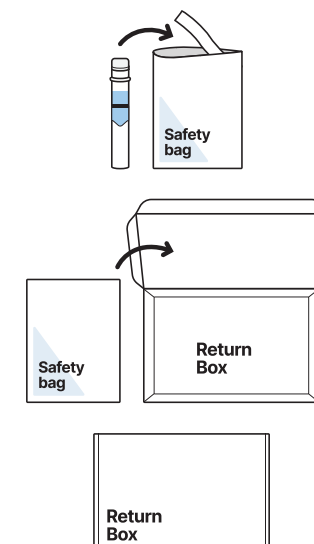10

Stop de safetybag in de retourbox.

- EN Put the safety bag in the return box.

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